

Focus on animal health

-  New gold standard for wound and skin hygiene
-  Fast and efficient application
-  Latest HOCL technology
-  Creates a low germ environment
-  Effective against bacteria, viruses and fungi
-  Free of antibiotics
-  Without alcohol, colorants and fragrances
-  Without burning or stinging (pH neutral)
-  For daily use



Available at your veterinary practice
or from your veterinarian.



Exclusively available
at the vet



**Efficient wound
management
for your pet**

Hoof, Wound & Skin Solution / Hydrogel

Sanocyn^{forte} Solution is ideal for rinsing and cleansing hoof, skin and wounds. The solution physically cleanses when generously flushing the affected area, thereby simultaneously reducing the germ load and preventing infections.

Sanocyn^{forte} Solution

Rinsing and Cleansing

Quick disinfection

Daily wound care

Removes dirt and debris



Sanocyn^{forte} Hydrogel, with its adhesive consistency, is perfect for moist wound and skin care and can develop its disinfecting effect over a longer period of time. Sanocyn^{forte} Hydrogel adheres excellently, keeps the wound moist and forms a protective film, which can prevent new and secondary infections.



Sanocyn^{forte} Hydrogel

Protects wound area

Enfolds effect over a longer period of time

Continuous wound moisturization

Adheres to hard-to-reach areas

SanocynBiDirection System

How to use

1. Expose affected areas
2. Rinse generously by spraying with Sanocyn^{forte} Solution
3. Apply Sanocyn^{forte} Hydrogel to protect wound area
4. Repeat application several times a day

Preputial Rinsing Solution

The Sanocyn^{forte} Preputial Rinsing Solution for male dogs contains ingredients that dissolve and remove dead cells as well as secretion of the mucous membranes, urine residues and debris at the same time. Regular use can control inflammation of the foreskin. In addition, it counteracts bacterial growth through its disinfecting effect, which protects against re-infections and successfully prevents unpleasant odours or spontaneous secretion.

Sanocyn^{forte} Preputial Solution

Fights unpleasant odours

Prevents inflammation of the foreskin

Painless, uncomplicated application

Extremely well tolerated



How to use

1. Clean the opening of the foreskin with a wipe soaked in Sanocyn^{forte} Preputial Rinsing Solution
2. Hold the foreskin and insert the applicator between the foreskin and the penis, bring in the Rinsing Solution
3. Massage to distribute the Rinsing Solution in the fold of the foreskin
4. Let the fluid drain off
5. Use for 3 to 5 consecutive days



Ear Cleaner & Eye Cleaner Solution

Sanocyn^{forte} Ear Cleaner Solution is a product for safe, easy and efficient cleaning of animal ears. Dirt and impurities are flushed out, irritation is relieved, bacteria, viruses and fungi are targeted.

Sanocyn^{forte} Ear Cleaner Solution

Routine hygiene and cleaning

Oil- and painfree and routinely application

Combating unpleasant odours

Dirt removal



How to use

1. Carefully guide the Ear Cleaner to the ear
2. Insert the solution into the ear and the ear canal by lightly squeezing the bottle
3. Gently massage the ear with the whole hand and carefully clean the ear with a swab or cotton ball
4. Application can be repeated several times a day

Sanocyn^{forte} Eye Cleaner Solution is a product for the gentle cleaning, care and soothing of the sensitive eye area of animals. Irritations are relieved and foreign bodies, dust and dirt can be rinsed out. The Sanocyn^{forte} Eye Cleaner can be used to safely and gently prevent discolouration under the eyes.



Sanocyn^{forte} Eye Cleaner Solution

Cleansing of the eye area

Soothing irritation

Rinsing out foreign bodies

Removal of tear fluid

How to use

1. Moisten one cotton pad or cloth per eye well with the solution
2. Clean the affected area from the inside out
3. Flushing out foreign objects: flush out with plenty of liquid by applying gentle pressure to the bottle
4. Repeat application several times a day if needed